



Application Instructions

PRODUCT: Active Compression Knee Sleeve
w/Patella Gel Support &
Flexible Stays

SKU: 308X

- Fits left or right knee
- Pull-on compression sleeve
- Patella gel stabilizer for maximum support



Order #	Size	Knee Circumference
3083	Small	12" - 14"
3085	Medium	14" - 16 ^{1/2} "
3087	Large	16 ^{1/2} " - 19"
3088	X-Large	19" - 21"
3089	2X-Large	21" - 23"

How to Apply

1. Pull the support up over the knee and center on the kneecap.
2. The support should fit snug but not so tight to restrict circulation.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling

