



Application Instructions

PRODUCT: Active Compression Knee Sleeve
SKU: 306X

- Fits left or right knee
- Pull-on closure
- Compression pain relief



Order #	Size	Knee Circumference
3063	Small	11" - 13"
3065	Medium	13" - 16"
3067	Large	16" - 19"
3068	X-Large	19" - 23"
3069	2X-Large	23" - 26"

How to Apply

1. Pull the support up over the knee and center on the kneecap.
2. The support should fit snug but not so tight to restrict circulation.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling

