



Application Instructions

Product: Deluxe Tri-Panel Knee Immobilizer

Model: 00XX

- Tri-Panel design for more anatomical fit
- Movable sidepanels for medial and lateral stay positioning
- Three contoured posterior stays
- Two medial and two lateral stays



Order #	Size	Length
0012	Universal	12"
0014	Universal	14"
0016	Universal	16"
0020	Universal	20"
0022	Universal	22"
0024	Universal	24"

SIZING: Universal design fits a range of sizes, size based on height/length.

HOW TO APPLY: The tri-panel style has adjustable side panels to fit a wide range of sizes and can be angled for a tapered fit. Angling the panels will improve the fit for wider thighs or thinner calves. The medial and lateral stays should still be aligned with the sides of the leg. Fully attach the hook on the side panels to the base. Place immobilizer behind knee using the contoured posterior stays as a guide for proper positioning. The inverted portion of the stays should be placed in the popliteal area (directly behind the knee). Close the panels of the immobilizer. With the immobilizer in correct position, start securing the straps immediately above and below the patella first. Work from the bottom upwards, closing the additional straps. Excess strap length may be trimmed to fit. Re-adjust if necessary.

CARE INSTRUCTIONS: Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION: Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L1830
AdVanced Orthopaedics, Inc.
326 Main Street Harleysville, PA 19438
800.270.7074
info@advanced-orthopaedics.com

This QR code can be scanned by your smart phone and directs you to the Product Detail Page.

