



Application Instructions

Product: Cervical Collars**Models: 200X- Premium, 20X0 Universal**

- Styled to give uniform support.
- Medium-firm density foam covered with tubular knit fabric.
- Hook and loop closure.
- Height refers to measurement under the chin.



Order #	Size	Length	Height
2003	Small	12"	2 ½"
2005	Medium	15"	3"
2007	Large	18"	3 ½"
2008	X-Large	21"	4"
2000	Universal	21"	2½"
2030	Universal	21"	3"
2035	Universal	21"	3½"
2040	Universal	21"	4"

HOW TO APPLY: Detach back hook and loop closure strap. Center the low contour of the brace underneath the chin. Wrap brace around neck. Fasten hook and loop contact closure and secure to a snug fit. Readjust if necessary. You should be able to place two fingers width between the collar and your neck to ensure collar is not too tight. Wear as prescribed by physician or caregiver.

CARE INSTRUCTIONS: Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION: Patients should be instructed to inspect their neck and this product before and after each use. Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L0120