



Product: Abdominal Binder

Models: 250X

- Available in 3 and 4 panels.
- Form-fitting, breathable abdominal binders conform to different patient shapes for improved compression.
- Made of soft foam and high quality elastic.
- Hook and loop closure.



Order #	Size	Waist Measurement
2501	3 Panel (9" Height)	30" - 45"
2502	3 Panel (9" Height)	45" - 62"
2503	4 Panel (12" Height)	30" - 45"
2504	4 Panel (12" Height)	45" - 62"

SIZING: Measure around waist at widest point.

HOW TO APPLY:

Apply in a standing or laying position. Open the binder to its fullest length with the care tag on the inside. Center the height of the abdominal binder at the navel with the seam centered in the back. The hook should be on the right side. Stretch both sides and wrap the sides without the hook around the body first. Secure the hook onto the loop portion. Support should fit snug but not too tight. Readjust if necessary.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A4466