



**Product: Neoprene Calf Sleeve Support**  
**Model: 301**

- Neoprene coated with 4-way stretch nylon fabric.
- Designed for relief and protection against injured calf muscles.
- Provides warmth and equal compression.



Order #	Size	Calf Circumference
301	Small	13" - 14"
301	Medium	14" - 15"
301	Large	15" - 17"
301	X-Large	17" - 19"

**SIZING:** Measure around the center of the calf.

**HOW TO APPLY:**

Pull the support up over the foot and position on the calf just below the knee joint with the seam in back. The support should fit snug but not so tight to restrict circulation.

**CARE INSTRUCTIONS:**

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

**CAUTION:**

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**WARNING:**

Contact neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its use immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.

**SUGG HCPCS CODE: A9270**