



Application Instructions

Product: AIRPENE Knee Sleeve

Model: 30X-AP

- A unique perforated neoprene-like material that retains heat but not moisture.
- The perspiration generated by exercise flows through the AIRPRENE to keep the skin dry and comfortable.
- AIRPRENE helps the brace stay firmly in place by dissipating excessive moisture, eliminating the slipping and movement of the conventional neoprene brace.



Order #	Size	Knee Circumference
303-AP	Small	12½" - 14½"
305-AP	Medium	14" - 16"
307-AP	Large	15½" - 18"
308-AP	X-Large	17½" - 20"
309-AP	2X-Large	19½" - 22"

SIZING: Measure around the center of the knee with leg extended.

HOW TO APPLY:

Pull the brace over the knee with the opening centered on the kneecap. The support should fit snug but not so tight to restrict circulation.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A4466