

Application Instructions

Product: Elastic Slip-On Knee Support Model: 32X Open Patella, 33X Closed Patella

- Easy slip-on design.
- Designed to give compressive support to weak knees or after mild injury.
- Durable knitted elastic provides support with flexibility.
- Seamless construction will not irritate skin.
- Available in either open or closed patella design.
- 11" sleeve.



SIZING: Measure around the center of the knee with leg extended.

HOW TO APPLY:

Pull the support up over the knee and center on the kneecap. The support should fit snug but not so tight to restrict circulation.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A9270

Advanced Orthopaedics, Inc. 326 Main Street Harleysville, PA 19438 800.270.7074 info@advanced-orthopaedics.com