



Product: Visco-Gel Silicone Knee Brace

Model: 33XX

- For tender knees resulting from sprains and strains.
- Flat knitted, four-way stretch elastic provides comfortable moderate compression.
- Contoured silicone ring redistributes patella pressure.
- Two medial and two lateral spiral steel stays.



Order #	Size	Knee Circumference
3333	Small	12" - 13½"
3355	Medium	13 ½" - 15"
3377	Large	15" - 16 ½"
3388	X-Large	16½" - 18"
3399	2X-Large	18" - 19 ½"

SIZING: Measure around the center of the knee with leg extended.

HOW TO APPLY:

Pull knee support up over knee. Position until the kneecap is surrounded by the visco-gel insert. Support should fit snugly but not so tight it digs into skin.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L 1800