

## Product: EZ Pull Adjustable Back & Abdominal Support

### FEATURES:

- Reduces stress and strain on your lower back
- Lightweight, breathable material & universally-sized
- Fully adjustable pulley fitting system
- Perfect for everyday use and can be worn under or over clothing

#### Order #

450

#### Location

Universal Fit



### HOW TO APPLY:

1. Place the EZ Pull handle towards the back of the brace by the pulley system.
2. While standing, position the pulley system on the middle of your back.
3. Place the left side fabric fastener over your stomach and then place the right side fabric fastener over your stomach.
4. Make sure the panel is tight enough to not fall off and then grab the EZ Pull handle and pull forward.
5. Once you have achieved the desired support, place the EZ Pull handle on the fabric fastener.
6. Loosen in the same way.



**CARE INSTRUCTIONS:** The rigid components can be wiped clean with a damp cloth. Soft goods can be hand washed in warm water with a mild detergent. Triple rinse and air dry. Avoid twisting the material to remove water.

*Note : Your practitioner is familiar with your specific medical condition and is using this product as part of your total care. Make sure to follow their directions exactly, even if they do not match the directions below.*

**CAUTION:** The application instructions listed above are intended for the use of qualified professionals only. It is not recommended for patients attempt to apply or adjust this device unless otherwise instructed to do so by an orthopedic professional. Any questions regarding further adjustment or frequency/ duration of wear should be directed to your orthopedic professional, as they know the individual condition for which this device was prescribed.

**SUGGESTED HCPCS CODE:** L0626