



**Product: Universal Wrap Around Knee Brace**  
**Model: 600**

- Excellent choice for those patients who are not able to pull a knee sleeve over their feet.
- Unique wrap design fits a wide range of sizes.
- Two medial and two lateral spiral steel stays.
- Universal buttress pad for easy adjustment and exact fit.



Order #	Size	Knee Circumference
600	Universal	12" - 23"

**SIZING:** Measure around the center of the knee with leg extended.

**HOW TO APPLY:**

Best applied when in standing position. Center opening of brace on the kneecap. Wrap the top thigh band around leg to desired compression. Fasten by attaching hook and loop closure to the support base. Repeat with the bottom calf band. Re-adjust thigh band if necessary. Ensure support is not too tight.

**CARE INSTRUCTIONS:**

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

**CAUTION:**

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**WARNING:**

Contact neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its use immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.

**SUGG HCPCS CODE: A4466**