

# **Application Instructions**

## Product: Suspension Sleeve Model: 620

- Perfect addition for patients that wear hinged knee braces.
- Universal sizing.
- Lightweight cotton/lycra sleeve helps prevent braces from sliding and moving.
- Neoprene lining provides extra comfort to patient.

Order # Size



### PREPARATION:

The Suspension Sleeve is a universally sized item and requires no trimming/adjustment before wear. The suspension sleeve can, however, be trimmed to a specific length/width for a more comfortable fit if needed.

#### HOW TO APPLY:

To Prevent Brace Migration: Once the knee brace is properly fitted and adjusted, wrap the suspension sleeve over the brace, using the thumb loops on the sleeve to achieve the appropriate tension. Secure the sleeve to itself via the velcro provided.

To Prevent Skin Irritation: Wrap the suspension sleeve around the desired area before the knee brace is applied, and secure the sleeve itself via the velcro provided. To be used in this manner, the supsension sleeve is most commonly applied to the thigh, the top of the calf, or directly over the knee itself. Once the suspension sleeve is properly secured, apply the knee brace over the sleeve as usual.

#### CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

#### CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

#### WARNING:

Contact neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its use immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.

SUGG HCPCS CODE: L2397 (PDAC Reviewed)

Advanced Orthopaedics, Inc. 326 Main Street Harleysville, PA 19438 800.270.7074 info@advanced-orthopaedics.com