

Products: The Weave™ Series
Models: 7X3

- Uniform Compression & Lightweight.
- Single Pull design.
- Rigid panels provide Posterior and Anterior support.
- Quick pull 6:1 mechanical advantage compression system.
- Breathable mesh belt.
- Aerospacer fabric-moisture wicking and breathable material.
- Patient friendly especially for arthritic patients.
- Ability to wear under clothes.



Weave™ 27 Order#	Weave™ 31 Order #	Weave™ 77 Order #	Size	Waist Circumference
733	743	773	Small	28" - 32"
735	745	775	Medium	32" - 36"
737	747	777	Large	36" - 40"
738	748	778	X-Large	40" - 46"
739	749	779	2XLarge	46" - 50"
739-9	749-9	779-9	3XLarge	50" - 56"
739-99	749-99	779-99	4XLarge	56" - 62"

SIZING: Measure around waist at widest point.

HOW TO APPLY:

Apply in a standing position. Open the strap and center the posterior panel of the brace in the middle of the back so the bottom is resting on the top of the buttocks. Wrap around the front closure and secure the velcro on the anterior panel. Pull the overlapping webbing strap out and up until snug, secure the velcro closure over the top of the front. Support should fit snug but not too tight. Readjust if necessary.

CARE INSTRUCTIONS:

Remove the rigid panels and fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, swelling, or inflammation.

SUGG HCPCS CODE: For 733 - L0627 and L0642
 For 743 - L0631 and L0648
 For 773 - L0637 and L0650