



Application Instructions

Product: Sport Lite Knee Brace

Model: 81X

- Enclosed neoprene tubular donut buttress.
- Two medial and two lateral spiral steel stays.
- Tapered in back for comfortable fit.
- Provides warmth and equal compression.



Order #	Size	Knee Circumference
813	Small	12½" - 14½"
815	Medium	14" - 16"
817	Large	15½" - 18"
818	X-Large	17½" - 20"
819	2X-Large	19½" - 22"
820	3X-Large	21" - 24"

SIZING: Measure around the center of the knee with leg extended.

HOW TO APPLY: Unfasten all straps and closures. Pull the brace over the knee with the opening centered on the kneecap and with the medial/lateral flexible stays aligned with the sides of the knee joint. Secure straps beginning with the top strap. Feed straps through rings and secure the hook and loop (Velcro-like) closure. Ensure support is not too tight.

CARE INSTRUCTIONS: Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION: Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

WARNING:

Contact neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its use immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.

SUGG HCPCS CODE: A4466