

Products: Transformer² Back Brace
Models: 80XX

- Double Pull design
- Rigid panels provide Posterior and Anterior support
- Quick double strap pull controls your support level
- Breathable mesh belt
- Easy to pull on and off
- Patient friendly, ensures greater patient compliance
- Produces intracavitary pressure to reduce load on intervertebral discs



Order #	Size	Back Panel Height	Side Panels
8027	SM - XL	10"	No
8031	SM - XL	14"	No
8037	SM - XL	14"	Yes

SIZING: Measure according to waist circumference.

Waist Circumference:

Small	26" - 32"
Medium	32" - 38"
Large	38" - 46"
XLarge	46" - 52"
2XL	52" - 60"


HOW TO APPLY:

To change the size of the Transformer², pull Velcro panel and lay flat with back of posterior panel facing up. Peel up the right or left side panel connectors and adjust to proper fit. The inside of the panel is marked (S, M, L, XL) on four semicircles. The Transformer adjusts to a wide range of sizing options along the four semicircles. Sizing is closely related to waist circumference. Once patient is sized, stretch the Transformer² out fully across the patients back and align brace in the center of back. Secure brace by overlapping the left panel then the right Velcro closure. Pull tension tabs until desired level of support is achieved and secure tabs to front panels.

CARE INSTRUCTIONS:

Remove the rigid panels and fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, swelling, or inflammation.

SUGG HCPCS CODE:

For 8027 - L0627 and L0642

For 8031 - L0631 and L0648

For 8037 - L0637 and L0650